

Limbal Relaxing Incisions for Dummies

Donnenfeld Nomogram- “DONO”

Eric Donnenfeld, MD

- 0.50 D: 1 incision 1 and a half clock hours
- 0.75 D: 2 incisions 1 clock hour
- 1.50 D: 2 incisions 2 clock hours
- 3.00 D: 2 incisions 3 clock hours

- A little more for against the rule and younger patients.
- A little less for older patients.
- Personalize your nomogram

This resource is intended for use by physicians and other healthcare professionals involved in patient care. It is not intended and should not be construed as medical advice, nor is it intended to replace sound clinical judgment in the delivery of healthcare services. All medical and clinical data contained or made available in this resource is intended to supplement the knowledge of physicians and other healthcare professionals involved in patient care. The absence of a warning for a given procedure, technique or suggestion contained in this resource should not be construed to indicate that such procedure, technique or suggestion is safe, appropriate or effective in any given patient.